

Operation Instructions

- 1.) Use switch on motor to raise and lower platform. (Switch for carrier lights is under black cap on motor.)
- 2.) Secure chair to platform using the three ratchet straps provided. See tie down instructions below.

Note: Air shocks may be required...periodically check and tighten all handles, bolts, roll pins, and screws...oil all parts on a regular basis.

Securing the Chair to the Carrier

- 1.) To operate ratchet straps: insert loose end of strap into groove in roller part of ratchet buckle. Then ratchet it down using handle on buckle. To *release* ratchet strap: pull spring loaded lock in ratchet handle and open buckle until it is locked in open position. Strap will then come loose.
- 2.) Hook one strap into front loop on platform. Wrap strap around tiller post of the scooter (or any accessible part of the power wheelchair frame.) Then hook the other end of the strap into the same loop. Tighten down strap securely.
- 3.) The second strap is hooked onto the first set of side loops. Hook one end of the strap into the loop, across the floorboard of the scooter (or around the front wheel posts of a power wheelchair), and then hook the other end into the opposite loop. Tighten strap down.
- 4.) The third strap attaches to the second set of side loops. Hook one end of the third strap into the loop. Wrap the strap around the seat post of the scooter (or through and around the rear wheels of the power wheelchair), and hook the other end into the opposite loop. Tighten strap down.



CAUTION: Do not over-tighten straps or damage to chair could result. Do not attempt to modify your carrier in any way... **this will void your warranty.**